
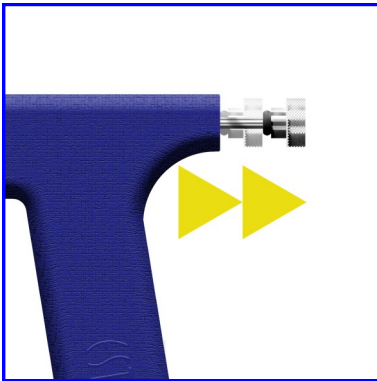


# Estelle Classic


## Ear Piercing System Operating Instructions

BRINGING YOU THE BEST IN PROFESSIONAL EAR PIERCING


Before use, prepare the piercing instrument by inserting either the mini or regular size adapter. To change the adapter, take the instrument in the un-cocked position and grip the adapter firmly between your thumb and first finger. Pull the plunger back to the cocked position, pulling the adapter at the same time to remove it. Insert the required size adapter into the plunger shaft and make sure it is fully seated.

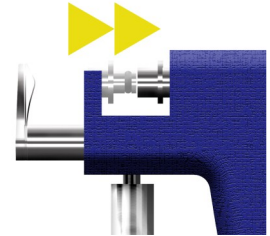


Pull plunger back into the cocked position.

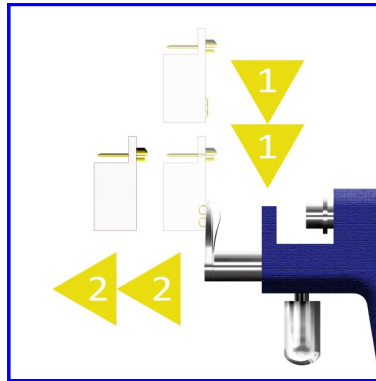


Slide the clasp down into the clasp holder and carefully pull away the plastic mount.

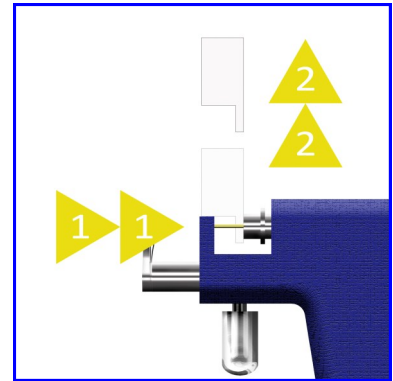


Insert the stud into the adapter and carefully pull straight up to release from the plastic mount.

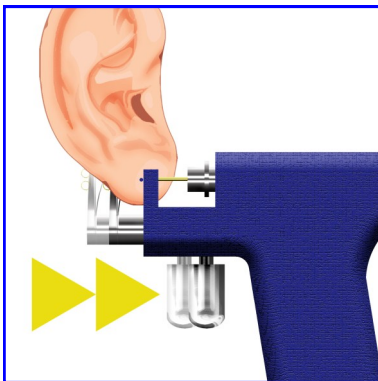

Place the ear between the clasp holder and the stud, align the point with the mark on the ear and slowly squeeze the trigger until the ear is held securely.

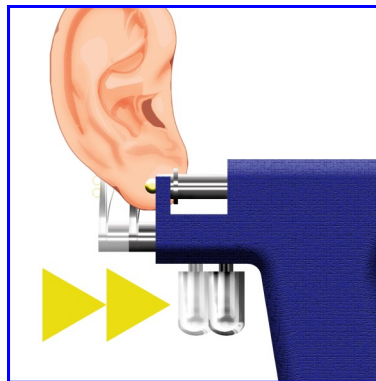

Keeping correct alignment, continue to squeeze the trigger until the stud is released and pierces the ear.

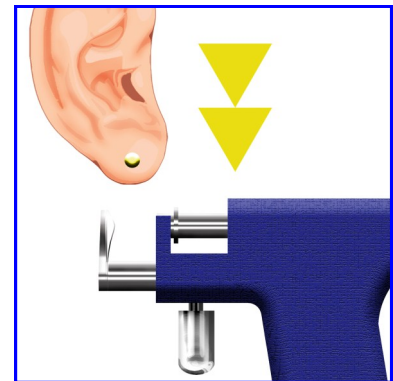

Carefully release the trigger to the fully open position and lower the instrument straight down from the ear.

[estelle.uk.com](http://estelle.uk.com)

ESTELLE  
— ENGLAND —